

INFANT IO LEG

Directions for Use



TABLE OF CONTENTS

Instructions for Use	3
Care and Maintenance	3
Replacement Parts	4

INFANT INTRAOSSEOUS (IO) LEG

The Infant IO Leg (365-00201) is a full-size infant right lower leg that can attach and articulate to the Infant Care Doll manikin or may be used as a stand-alone teaching model.


It is specifically designed for training professionals in the practice of intraosseous infusion on infants.

The manufacturing excellence of this trainer should provide many sessions of training when reasonable care and maintenance is practiced.

INSTRUCTIONS FOR USE

1. Unplug the drain in the heel of the right foot. (Fig. 1)
2. Using a syringe, fill the leg with approximately 300cc of fluid. (Simulated blood is available).
3. Reinsert the plug in the heel of the foot.

Fig. 1

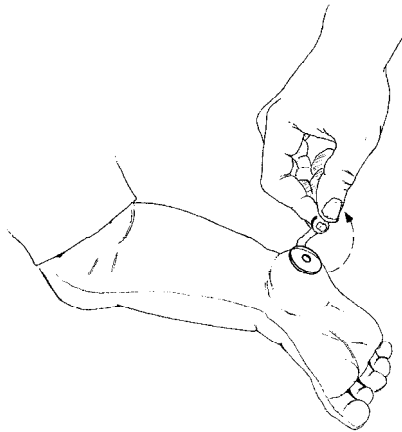

4. The IO Leg is now ready to practice intraosseous infusion.

CARE AND MAINTENANCE

1. Use on clean surface only. AVOID FELT TIPPED MARKERS, INK PENS, ACETONE, IODINE OR OTHER STAINING MEDICATIONS, AND PLACING THE SIMULATOR ON NEWSPRINT OR INKED LINES OF ANY KIND.
2. Clean with mild soap and water.
3. A general inspection should be conducted regularly.
4. Store properly between teaching sessions.

REPLACEMENT PARTS

The following replacement part numbers are valid for simulators with white skin tone only. Please contact our Customer Service Representatives for replacement part numbers if your manikin has a different skin tone.

Product Number:

365-01250

365-00450

365-03050

Description:

Infant IO Leg Mandrel

Infant IO Leg Skin Sleeve

IO Foot Insert

© Laerdal Medical, Gatesville TX. All rights reserved
1005314 rev B
November 22, 2006